# Dispenser DP 20

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Nr. 31868

[www.nouvag.com](http://www.nouvag.com)

**NOUVAG**<sup>®</sup>  
Dental and Medical Equipment

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Congratulations on your purchase of a NOUVAG AG product. Thank you for the confidence shown in our products.

Please consult the instruction manual for the use and maintenance of the device in order to ensure that it will function properly and efficiently for many years.

You will find the conformity statement and list of authorized service representatives attached.

- **Before operating, please read instructions carefully!**

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# TABLE OF CONTENTS

<b>1</b>	<b>Description</b>	<b>2</b>
1.1	Operation and Functions	2
1.2	Technical data	2
1.3	Operating Environment	2
1.4	Transport and Storage Environment	2
1.5	Icons	2
<b>2</b>	<b>Safety Measures</b>	<b>3</b>
<b>3</b>	<b>Shipping contents</b>	<b>3</b>
3.1	Dispenser DP 20 complete	3
<b>4</b>	<b>System Assembly</b>	<b>3</b>
4.1	Power supply connection	4
4.2	Footcontrol connection (optional)	4
4.3	Infiltration Accessory Connection	4
<b>5</b>	<b>Operation</b>	<b>5</b>
5.1	Starting the Dispenser DP 20	5
5.2	Regulation of the infiltration procedure	5
<b>6</b>	<b>Cleaning and Disinfection</b>	<b>6</b>
6.1	Control unit and foot control	6
<b>7</b>	<b>Maintenance</b>	<b>6</b>
7.1	Control Unit Fuse Replacement	6
7.2	Disposal	6
<b>8</b>	<b>Troubleshooting</b>	<b>7</b>
<b>9</b>	<b>Replacement parts and numbers</b>	<b>7</b>
<b>10</b>	<b>Servicecenter</b>	<b>8</b>

**1 Description**

**1.1 Operation and Functions**

The Dispenser DP 20 is a controllable pump designed for the transportation of fluids. The area of application of the Dispenser DP 20 is widespread. This device is used specially as Infiltration pump for the removal of fatty tissue. With proper application, the lipolysis procedure is simple, safe, cost effective and delivers optimal cosmetic results.

The Dispenser DP 20 can be used in the following fields:

- Laboratory/ Medicine
- Chemistry/ Pharmaceutical Industry
- Food- / Beverage Industry
- Photograph / Sewage

**1.2 Technical data**

Voltage: ..... changeable: 100 V~, 115 V~, 230 V~/ 50 – 60 Hz  
 Current consumption Max.: ..... 200 mA  
 Fuses (2 pieces): ..... T 1 A  
 Suction Pump Capacity: ..... 1l/h - 20 l/h  
 Dimensions: ..... 150 x 280 x 260 mm (H x B x T)  
 Weight: ..... 4.3 kg

**1.3 Operating Environment**

Relative Humidity: ..... max. 80%  
 Temperature: ..... 10 to 40°C  
 Barometric Pressure: ..... 800 to 1060hPa

**1.4 Transport and Storage Environment**

Relative Humidity: ..... max. 90%  
 Temperature: ..... 0 to 60°C  
 Barometric Pressure: ..... 700 to 1060hPa

**1.5 Icons**

: Attention: Consult accompanying documents

: Class II equipment

1275 : Conforms to EU standards

: Certified by the Canadian Standards Association (CSA) for Canada and USA

: Type BF applied part

: Old electrical and electronic equipment must be disposed separately and may not be included in regular domestic waste.

**IP68**

: Floor pedal watertight integrity

: Functional earth terminal

**2 Safety Measures**

Your safety, the safety of your team, and, it goes without saying, the safety of your patients is for Nouvag AG the first priority. It is therefore vital that the following measures be strictly observed:

- **The Dispenser DP 20 should be operated by qualified personnel only!**
- **Check the correct operation voltage on the rating plate!**
- **Responsibility for the use of accessories, parts or assemblies from other manufacturers rest solely with the user!**
- **Repairs are to be carried out by authorized NOUVAG AG service technicians only!**
- **NOUVAG AG cannot be held liable for any malfunction of the Dispenser DP 20, or performance failure and/or its designed or desired utility, nor can NOUVAG AG be held liable for any injuries to persons or animals, in any case when the Dispenser DP 20 is miss-used or not operated, applied or maintained in strict accordance with the user/owner instructions set out in the operating manual. In the event of any doubt or question, the user is to contact NOUVAG AG or its lawful representative for clarification or assistance!**
- **Do not use the unit near heat or open flame.**

English

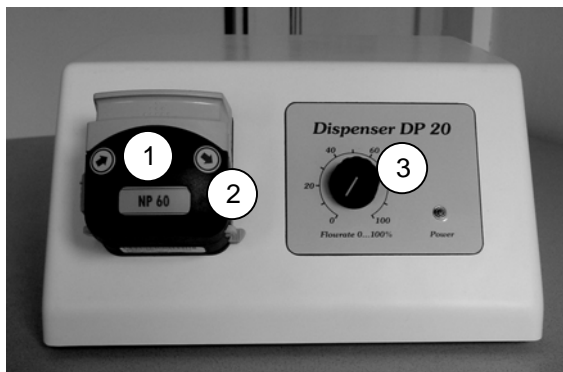
**3 Shipping contents**

**3.1 Dispenser DP 20 complete**

Art.-Nr.	Description	Units
4165.....	Control unit DP 20.....	1 piece
6022.....	Tubing set; disposable with plug-in cannula, sterile, length 4m.....	1 piece
1770.....	Hanger rod for liquid bottle.....	1 piece
<b>optional:</b>		
1486.....	Vario-pedal.....	1 piece
1410.....	Footcontrol On/Off.....	1 piece


**4 System Assembly**

Front panel

1. Pump head
2. Adjusting screws
3. Infiltration rate control knob

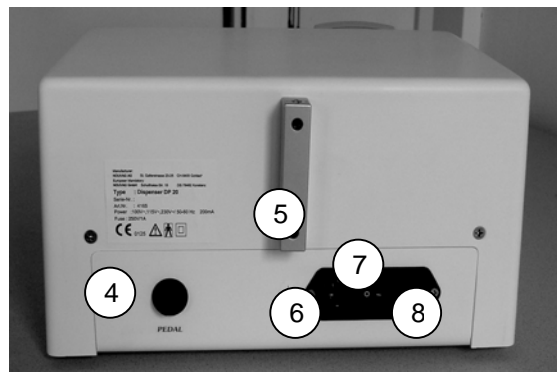Rear side

4. Footcontrol connector
5. Hanger rod support
6. Fuse Case
7. Mains switch
8. Mains Connector

### 4.1 Power supply connection

Before plugging the mains cable, check for correct operation voltage beside the mains switch. If the voltage is not according to the local power supply (see 7.1 Control unit fuse replacement), turn the gray fuse case to the correct voltage.

### 4.2 Footcontrol connection (optional)

A foot control can be connected to the Dispenser DP 20 (on the back of the control unit-)

**Connect „On/Off“-pedal (Art.-Nr.1410) or Vario-pedal (Art.-Nr.1486) to the Dispenser DP 20 only !**

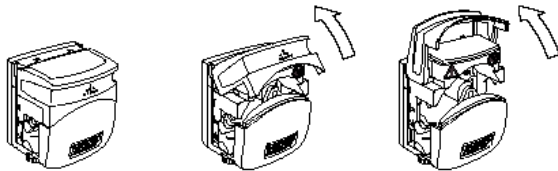
### 4.3 Infiltration Accessory Connection

**Switch off the device before connecting the infiltration accessory!**



1) Turn left and right adjusting screw on the pump head up to the right stop position (8.0).

2) Lift lid to open

3) Insert tube for infiltration . (→ Pay attention to flow rate  direction)

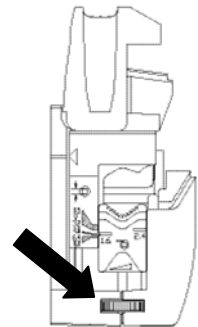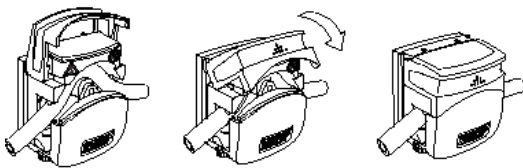4) Put lid down to close.

- The insertion cannula must be on the right side (suction side of the peristaltic pump), insure that „pressure“ tube runs to infiltration needle, and „suction“ tube runs to solution bottle!
- Use the designed tubing set (Art.-Nr.6022) only !

5) Insert tube cannula and hang Infiltration bottle (not included) on the hanger rod (not included).

6) Open tube clamp.

7) Open air valve to insert cannula.

8) Attach infiltration needle to other end of tube.

**The system is operational!**

**5 Operation**

**5.1 Starting the Dispenser DP 20**

Press the „I/O“ On/ Off switch on the back panel of the control unit

→ The green power control light will illuminate.

**5.2 Regulation of the infiltration procedure**

- Do not open the pump head during operation.
- Do not touch rotatable parts during operation!

English

**Regulation of the pump without pedal**

Adjust proportionally the infiltration rate with the control knob. Do not connect the pedal in this case.

**Controlling the pump with Footcontrol On/Off (optional)**

The Dispenser DP 20 can be controlled with Footcontrol On/Off. Please pay attention to the following important points:

The Footcontrol „On / Off“-functions only, when the mains switch „I/O“ is turned on and the Power control light illuminates. To start the infiltration procedure push the Footcontrol down. The pump speed can be always regulated with the control knob.

**Controlling the pump with Vario-pedal (optional)**

The Dispenser DP 20 can be controlled with Vario-pedal. Please pay attention to the following important points:

The Vario-pedal functions only, when the mains switch „I/O“ is turned on and the Power- control light illuminates. With the pedal, pump speed can be variably controlled:

<i>Pedal</i>	... not pressed: .....	<i>Pump off</i>
	... lightly pressed.....	<i>Pump runs slowly</i>
	...fully pressed: .....	<i>Pump speed max. as indicated by setting on control unit</i>

## 6 Cleaning and Disinfection

Please pay attention to the following important points for the maintenance of the material:

- Do not use dissolving agents for cleaning
- Sterilized material should be stored and tagged with sterilizing date!

### 6.1 Control unit and foot control

The control unit and the floor pedal do not come into contact with patients. Only clean the surface of the unit, do not use harsh cleaners or solvents for cleaning. Use 80% ethyl alcohol or microbiologically effective disinfectants.

**The Control Unit should be wiped with a clean, damp cloth only. No spray cleaners or disinfecting agents should be employed, as the Control Unit face is not watertight!**

## 7 Maintenance

### 7.1 Control Unit Fuse Replacement

A blown Fuse can be easily replaced by the operator. It is located on the back of the Control Unit:

- 1) Disconnect mains.
- 2) With screwdriver open fuse cover.
- 3) Replace defective fuse T 1A.
- 4) With Screw driver close gray fuse case.
- 5) Check voltage on fuse case.
- 6) Reconnect mains.

### 7.2 Disposal

Disposal of device, components and accessories must strictly conform to local laws and regulations as set out by the relevant authorities.

## 8 Troubleshooting

<b>Problem:</b>	<b>Cause:</b>	<b>Solution:</b>	<b>Refer: Operating Manual</b>
<b>Device does not operate at all: → Power-control light does not illuminate</b>	Control Unit not turned on	Turn Main Switch «I/O» on the back of the unit to position	5.1 Starting the Dispenser DP 20
	Mains not connected	Connect mains to Control Unit	4.1 Power supply connection
	Defective Fuse	Check fuse and replace if indicated	1.2 Technical data 7.1 Control Unit Fuse Replacement
	Wrong operating voltage	Check mains voltage output	1.2 Technical data 4.1 Power supply connection
<b>Pedal fails to function: → Power-control light illuminates</b>	Pedal not connected	Plug Floor Pedal Cable into rear of Control Unit	4.2 Footcontrol connection
	Improper use	Refer to Operating Manual	5.2 Regulation of the infiltration procedure
<b>Pump fails to function : → Power-control light illuminates</b>	Trip plate not pressed down	Press plate down to control infiltration procedure with the pedal	5.2 Regulation of the infiltration procedure
	Infiltration speed too low	Increase the pump speed with control knob	5.2 Regulation of the infiltration procedure
	Suction Set incorrectly attached	Attach Suction Set correctly (check direction!)	4.3 Infiltration Accessory Connection
	Adjusting Screw branches off the Infiltration tube	Screw up the Adjusting Screw till the right stop position	4.3 Infiltration Accessory Connection
	Improper use	Refer to Operating Manual	5.2 Regulation of the infiltration procedure

In the event a problem cannot be solved, contact dealer or authorized service technicians, as listed on the last page of your Operating Instruction Manual.

## 9 Replacement parts and numbers

<b>Description</b>	<b>Art. No.</b>
Footcontrol On/Off .....	1410
Vario-pedal .....	1486
Hanger rod for liquid bottle .....	1770
Tubing set for infiltration pump; disposable with plug-in cannula, sterile, length 4m .....	6022

For ordering other parts, our customer service personnel are pleased to assist.

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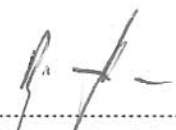
## Electromagnetic compatibility (EMC)

Remark:

The **Product** subsequently referred to herein always denotes the Dispenser DP 20.

Changes or modifications to this product not expressly approved by the manufacturer may result in increased emissions or decreased immunity performance of the product and could cause EMC issues with this or other equipment. This product is designed and tested to comply with applicable regulations regarding EMC and shall be installed and put into service according to the EMC information stated as follows.

### WARNING

Use of portable phones or other radio frequency (RF) emitting equipment near the product may cause unexpected or adverse operation.

### WARNING

The product shall not be used adjacent to, or stacked with, other equipment. If adjacent or stacked use is necessary, the product shall be tested to verify normal operation in the configuration in which it is being used.

### Compliant Cables and Accessories

### WARNING

The use of accessories, transducers and cables other than those specified may result in increased emissions or decreased immunity performance of the product.

The table below lists cables, transducers, and other applicable accessories for which the manufacturer claims EMC compliance.

**NOTE:** Any supplied accessories that do not affect EMC compliance are not listed.

Description	Length max.
Foot switch Vario	2.9m
Foot switch On/Off	2.9m

### Guidance and manufacturer's declaration – electromagnetic emissions

The Product is intended for use in the electromagnetic environment specified below. The customer or the user of the Product should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The Product uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Product is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

### Guidance and manufacturer's declaration – electromagnetic immunity

The Product is intended for use in the electromagnetic environment specified below. The customer or the user of the Product should assure that it is used in such an environment.

Immunity tests	IEC 60601 Test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	+/- 6 kV contact +/- 8 kV air	+/- 6 kV contact +/- 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	+/- 2 kV for power supply lines +/- 1 kV for input/output lines	+/- 2 kV for power supply lines +/- 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	+/- 1 kV differential mode +/- 2 kV common mode	+/- 1 kV differential mode +/- 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5 % $U_T$ (> 95 % dip in $U_T$ ) for 0,5 cycle  40 % $U_T$ (60 % dip in $U_T$ ) for 5 cycles  70 % $U_T$ (30 % dip in $U_T$ ) for 25 cycles  < 5 % $U_T$ (> 95 % dip in $U_T$ ) for 5 sec	< 5 % $U_T$ (> 95 % dip in $U_T$ ) for 0,5 cycle  40 % $U_T$ (60 % dip in $U_T$ ) for 5 cycles  70 % $U_T$ (30 % dip in $U_T$ ) for 25 cycles  < 5 % $U_T$ (> 95 % dip in $U_T$ ) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment.  If the user of the Product requires continued operation during power mains interruptions, it is recommended that the Product be powered from an uninterruptible power supply or a battery.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Note:  $U_T$  is the a.c. mains voltage prior to application of the test level.

**Guidance and manufacturer's declaration – electromagnetic immunity for not life support equipment**

The Product is intended for use in the electromagnetic environment specified below. The customer or the user of the Product should assure that it is used in such an environment.

Immunity tests	IEC 60601 Test level	Compliance level	Electromagnetic environment - guidance
			Portable and mobile RF communications equipment should be used no closer to any part of the Product, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.  <b>Recommended separation distance:</b>
Conducted RF IEC 61000-4-6	3 V rms 150 kHz to 80 MHz outside ISM bands	3 V rms 150 kHz to 80 MHz outside ISM bands	$d = 1,2\sqrt{P}$
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m 80 MHz to 2.5 GHz	$d = 1,2\sqrt{P}$ 80 MHz to 800 MHz  $d = 2,3\sqrt{P}$ 800 MHz to 2,5 GHz
			where $P$ is the maximum output power rating in the transmitter in watts (W) according to the transmitter manufacturer and $d$ is the recommended separation distance in metres (m).  Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey <sup>a</sup> , should be less than the compliance level in each frequency range <sup>b</sup> .  Interference may occur in the vicinity of equipment marked with the following symbol:  

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Fixed strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To access the electromagnetic environment due to fixed RF transmitters, and electromagnetic site survey should be considered. If the measured field strength in the location in which the Product is used exceeds the applicable RF compliance level above, the Product should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the Product.

b over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

**Recommended separation distances between portable and mobile RF communications equipment and the not life support equipment**

The Product is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Product can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Product as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2.5 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the higher frequency range applies.

Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

# Servicestellen / Servicecenter / Service / Assistenza tecnica / Centro de Servicio

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## GREAT BRITAIN

### S.R. Electromatics

511 Fulbridge Road  
Werrington  
GB - PE4 6SB Peterborough  
Great Britain

Phone: +44 (0)1733 571958  
Fax: +44 (0)1733 571958  
E-Mail: srelectromatics@hotmail.co.uk

## ASIA

### Associated Medical Supplies Co. Ltd.

Room 1201, Fo Tan Industrial Center  
26 Au Pui Wan Street  
Fo Tan, Shatin  
Hong Kong

Phone: +852 2604 9389  
Fax: +852 2694 0866  
E-Mail: general@amscl.com

- 
- D** Wir bitten Sie, bei Benötigung von Service, Reparatur oder Ersatzteilen, den Händler oder Vertreter Ihres Landes zu kontaktieren.
- GB** In case of service, maintenance and spare part questions – we kindly ask you to contact your dealer or representative in your area.
- F** En cas de demandes regardent le service technique, l'entretien ou pièces détachées, nous vous prions de contacter le revendeur ou représentant dans votre région.
- I** La preghiamo di contattare il suo rivenditore o il nostro rappresentante della sua zona, riguardo l'assistenza tecnica, la manutenzione o per richiedere pezzi di ricambio.
- E** Por favor contactar al distribuidor en su area, para preguntas concernientes a servicio, mantenimiento y repuestos.
-